


PMMC Preliminary Report Workflow

Launch MRG VPN




Log on to MRG VPN using your MRG credentials.

'MRG\' is NOT needed.

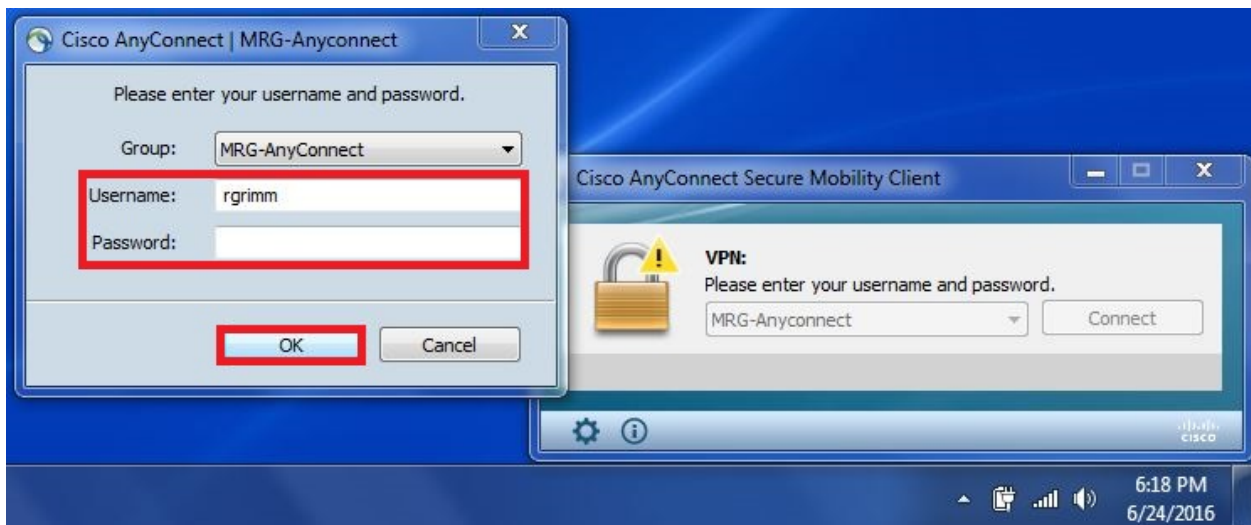

This will allow Synapse to run similar to being at MRG. You can access **RRMC, ACH, OUTPACS, MRG/SI, MRIC, MMC, HOA, and RVP** now no need to use SSL sites with the AnyConnect on.

Go to the Providence web portal using Internet Explorer

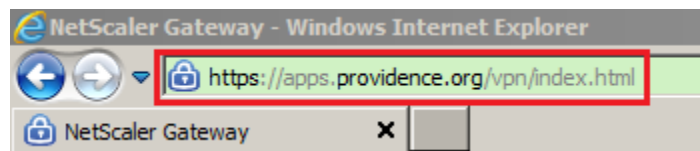

Enter your Providence credentials and press enter or 'Log On'

ACCEPTABLE USE AGREEMENT:

This system is the property of Providence Health & Services and is intended for authorized use only. Use of this system is subject to monitoring. Unauthorized use or access may be a violation of federal and state laws and regulations. Unauthorized users may be subject to criminal and civil penalties. Use of this system serves as your acknowledgement of and agreement to abide by applicable Providence Health & Services policies.


**Please log on to
continue.**

User name

Password

Log On

Click 'Phillips iSite'

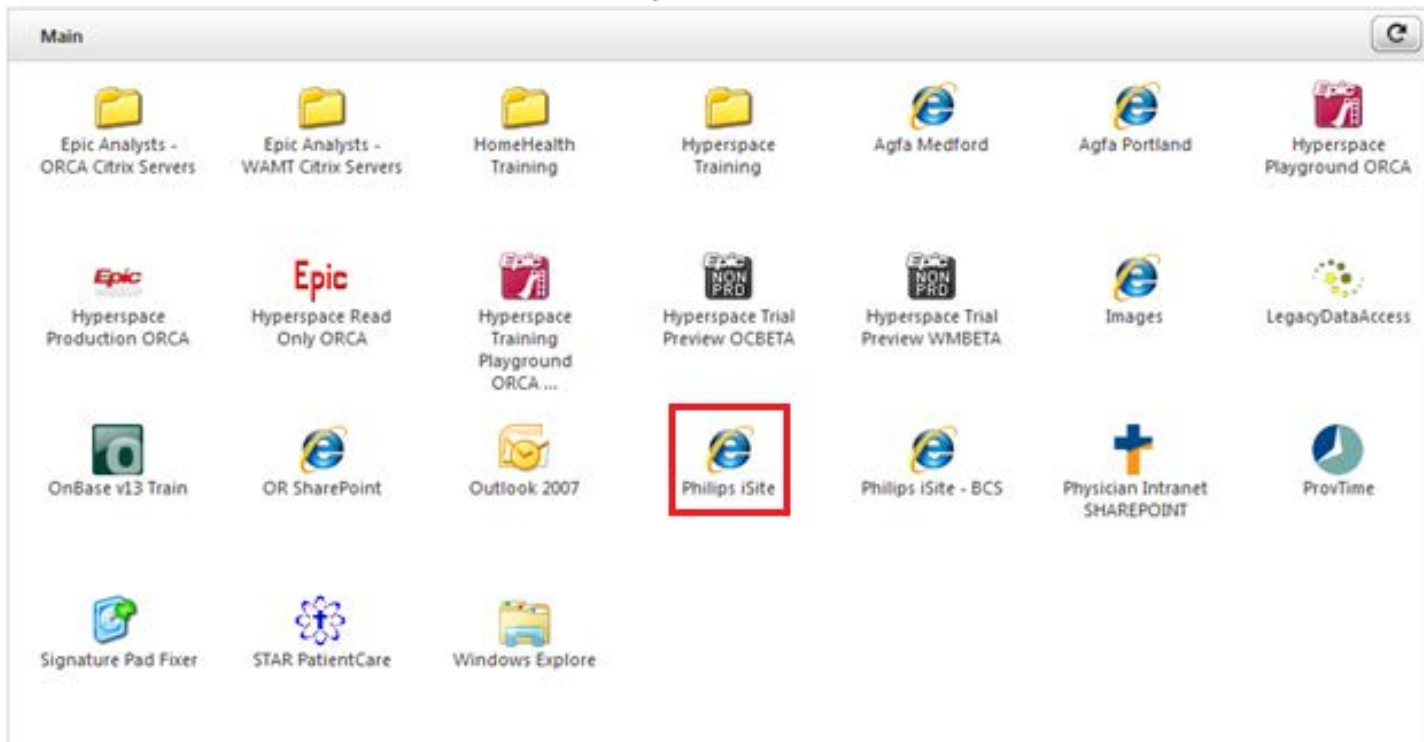

Enter your Providence credentials again and hit enter or 'Log in'



PHS OR iSite Medical Imaging 3.6.150

User Name

Password

Log on to
iSite

Workstation Location
Remote Access



Log In

Copyright © 2006-2013 Koninklijke Philips N.V. All rights reserved. **CE0044**
3.6.150.0 Date of Manufacture: Tuesday, December 03, 2013
Protected by U.S. Patents 6,553,141; 6,711,297; 6,734,880


1. Enter patient name by Last Name,First Name format and press 'enter'
2. Click appropriate patient record to open exam list
3. Double-click exam that you would like to open

Close All	Patient Name	
+ Query	test, scissor	
+	test, scissors	00000
CR - CR; Acc: 34758; Referring: Unassigned, Ordering Provider		

Single-click on the clipboard icon

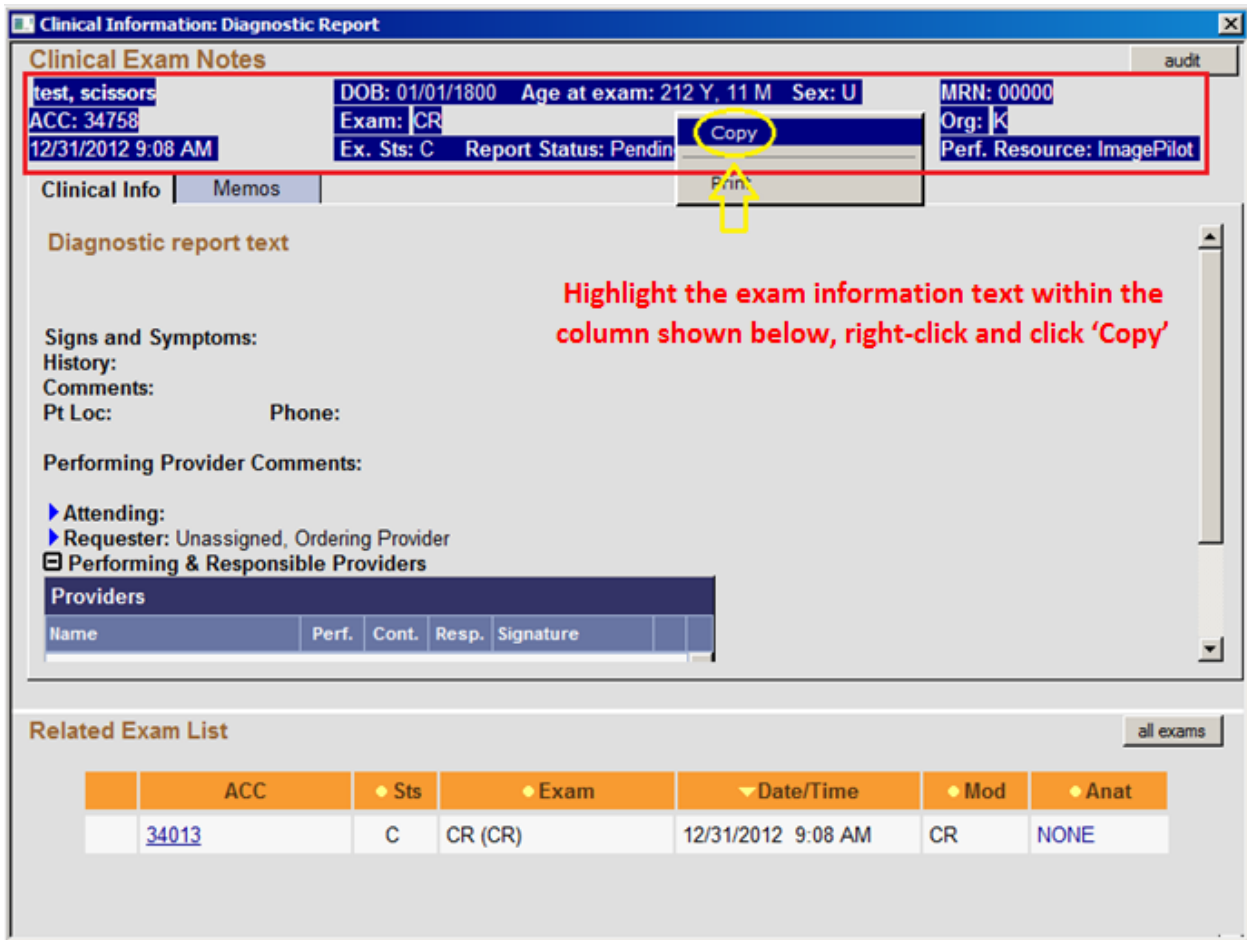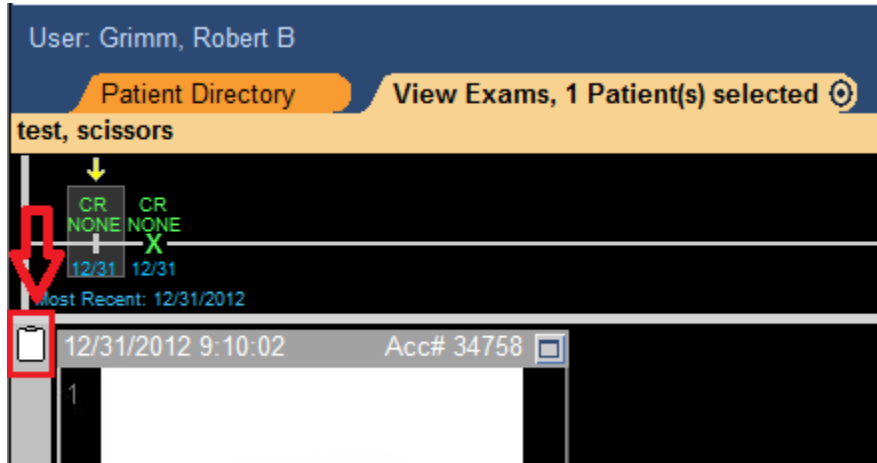

Open Prelim Report Form

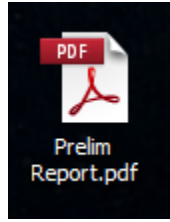



MEDFORD RADIOLOGY GROUP

Preliminary Radiology Report

For Assistance Call: 541-618-5800

Exam Information:

test, scissors DOB: 01/01/1800 Age at exam: 212 Y, 11 M Sex: U MRN: 00000 ACC: 34758 Exam: CR Org: K 12/31/2012 9:08 AM Ex. Sts: C Report Status: Pending Creation Perf. Resource: ImagePilot

IMPRESSION:

Test prelim report.

Dictated and Authenticated by


Paste the exam information into the appropriate field and complete the rest of the form as shown below

Please remember to include the site that images are from.

Example: PMMC


CONFIDENTIALITY STATEMENT, THIS REPORT IS INTENDED ONLY FOR THE USE OF THE REFERRING PHYSICIAN, AND ONLY IN ACCORDANCE WITH LAW, IF YOU RECEIVED THIS IN ERROR, CALL 541-618-5800]

Click the Print icon at the top of the Adobe window

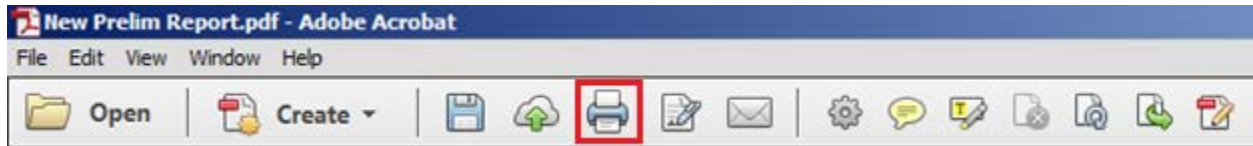


'MRG Reading Center Fax' should be automatically selected as the default printer, but if it is not, please choose it from your list.

Click 'Print'

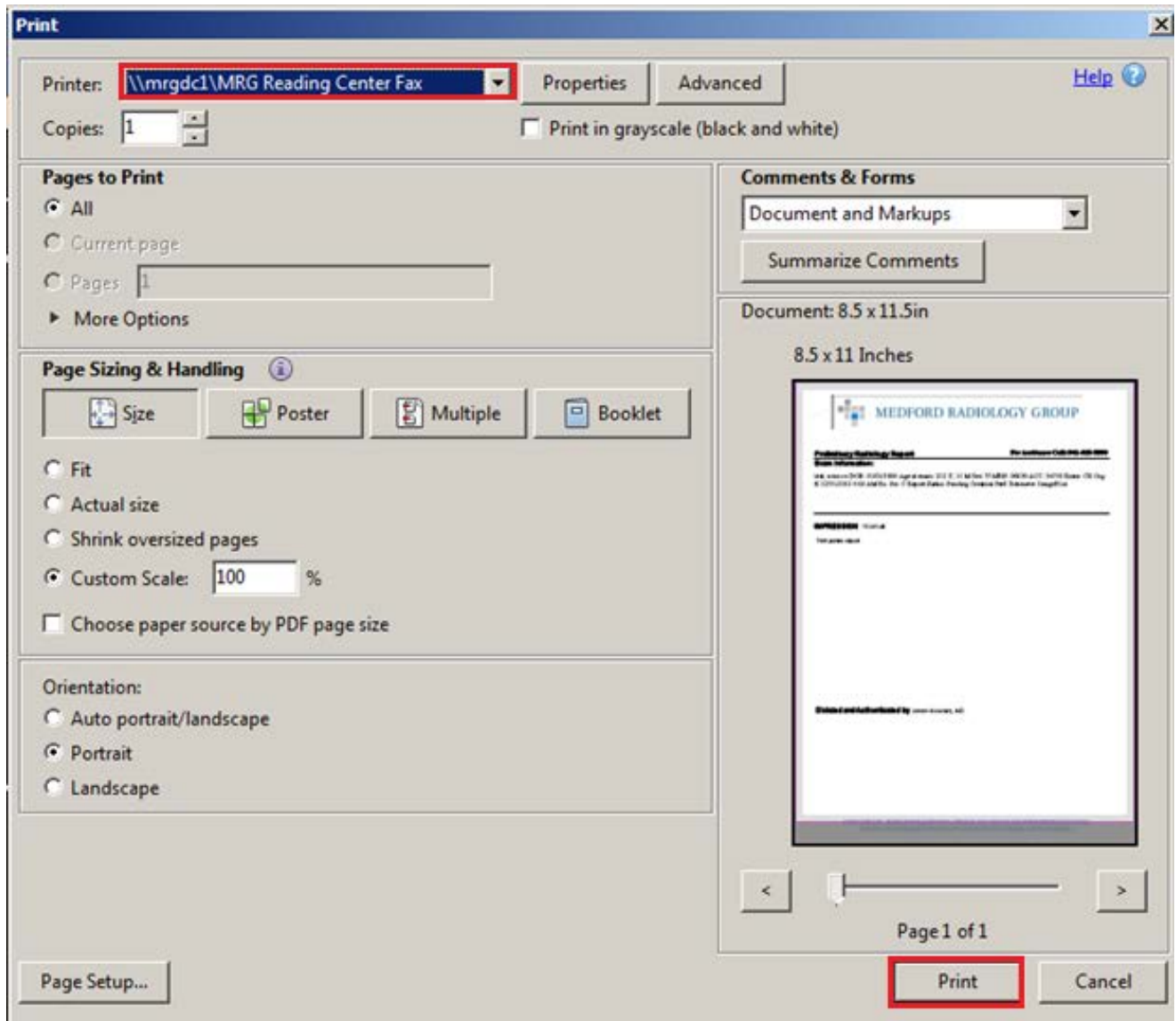


****Ensure that you choose the group -AACH, PMMC, or RPMC- rather than an individual destination****

**Selecting the appropriate group will fax your report to the ER, Radiology File Room, and secondary MRG fax machine
Click 'Send'**

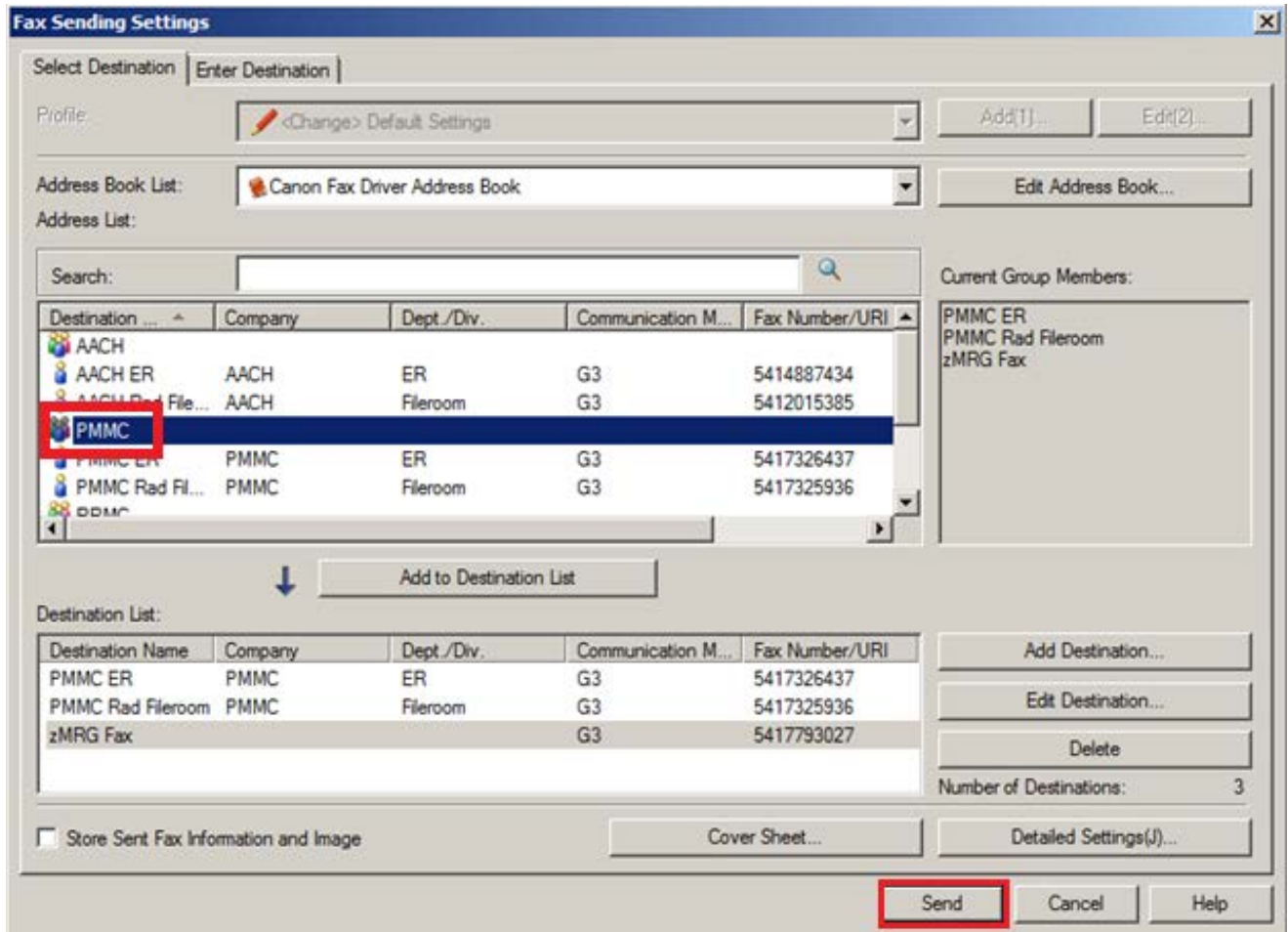

Confirm your selection by clicking 'Ok'

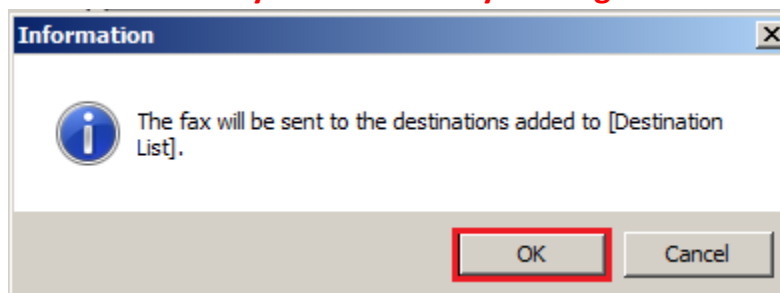

You are now finished! You may now close your report.